
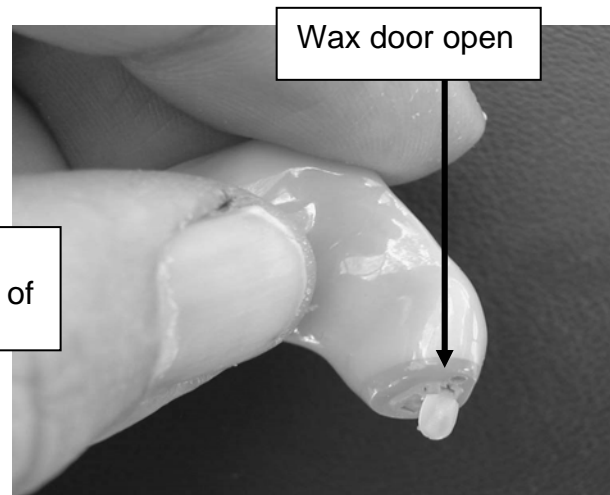


# Cleaning Your ITE (In The Ear) Hearing Aid with Wax Door

## How to keep your hearing aid working

Wax blocking the sound outlet of the hearing aid is a very common reason for aids to stop working. Sound simply cannot get out of the hearing aid and into your ear.

A simple cleaning procedure every time you change your battery will prevent this.

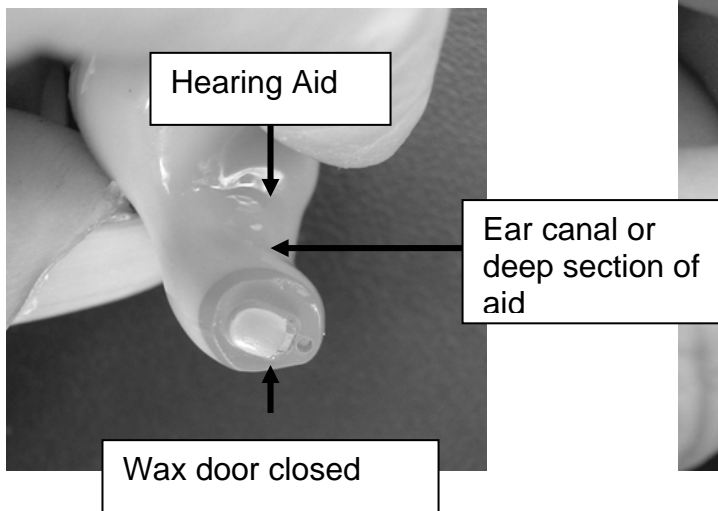

1. Locate the wax door, which collects wax. It may be cleaned with wax door open or closed. See below for details.
2. Hold the aid with the wax door pointing downward.
3. If door is open, **GENTLY** brush the door free of wax, as it easily snaps off.
4. To brush more vigorously, make sure the door is **CLOSED**.
5. If door comes open, close it gently.